

# ConvaQuip Ind., Inc.

## Manual

for

Model 724/724DAU

Bedside Commode



Fixed Arms



Shown with arms in open position

Swingaway Arms  
(aka Droparms)

Capacity 850 lbs.

Model 724DAU Swingaway Arms:

One or both arms can be lifted by pulling gently upward and out of cradle, allowing the arm to Swingaway and allowing a user to transfer onto or off of the bedside commode.

Once the transfer is made, swing the arm back into place (closed position) and make sure the arm is placed back into the cradle. Gravity will keep the arm in the correct position for use. Never use commode when arm/arms are in the open position.

**CAUTION: NEVER USE OR PUT WEIGHT ON THE ARM WHEN IN THE OPEN POSITION. THIS COULD CAUSE INJURY TO THE USER AND DAMAGE TO THE BEDSIDE COMMUNE.**

Also note that Swingaway Arms can be removed from the bracket entirely by pulling the arm upward and pushing the release button on the back of the arm.

:

### Cleaning Instructions:

- \* For infection control, clean all parts with one part bleach and 9 parts water. Rinse and allow to air dry or wipe with a clean, dry cloth.
- \* Clean rubber tips with alcohol to remove oil, grime and dirt
- \* Pail may be steam cleaned or use Beaucoup disinfectant (made by EcoLab) or equivalent.