



Shower Chair/Commode Instructions & Warranty

PRODUCTS: 800 GS, 1000 PS

CLEANING

All black surfaces are made from a Polypropylene material and must be cleaned with a non-abrasive cleaner.

The chairs are designed to be cleaned in an automatic cart washer; cleaning temperatures from ambient to 200 degrees for a period of up to 10 minutes. The chairs can be dried with saturated steam for up to 3 minutes. Collection buckets must be removed during this process.

ADDITIONAL SAFETY INSTRUCTIONS

All 4 casters **MUST** be locked during use and patient transfer.
Casters are locked when the lever is in the down position

Before moving patient while in Commode, ensure that all casters are unlocked.

For safe control and movement of patient, place both hands on the back handles.
Chairs hold up to 1,000 lbs or 454 Kg.

WARRANTY

** Lifetime warranty on stainless steel frame ** 6 Months on casters ** All other parts and accessories 1 year **

Please contact CONVAQUIP at 800-637-8436, for any loose, damaged or missing part and discontinue the use of the chair until the repair has been done.