

1. MFG Direct 30000 Series treatment tables

The MFG 30000 Series treatment tables are intended for healthcare professionals in hospitals and clinics. The treatment tables are designed to be used in a range of physiotherapy treatments while patients are on the table top. The series contains several models: with electric height adjustment.

This document gives instructions for operating and maintaining MFG treatment tables. Please familiarize yourself with these instructions before using the table. Use the device only as described and for the specified applications. Store these instructions in an appropriate way, making sure that the instructions are available to all possible users throughout the life of the device.

The supplier will make available on request circuit diagrams, component parts list, descriptions, calibration instructions, or other information which will assist the User's appropriately qualified technical personnel to repair those parts of equipment which are designated by the manufacturer as repairable.

IMPORTANT! To avoid injury, follow the instructions given in this document

1.1 Description of parts

The basic sections of the treatment table are shown in Figure 1. These sections are similar in other models.

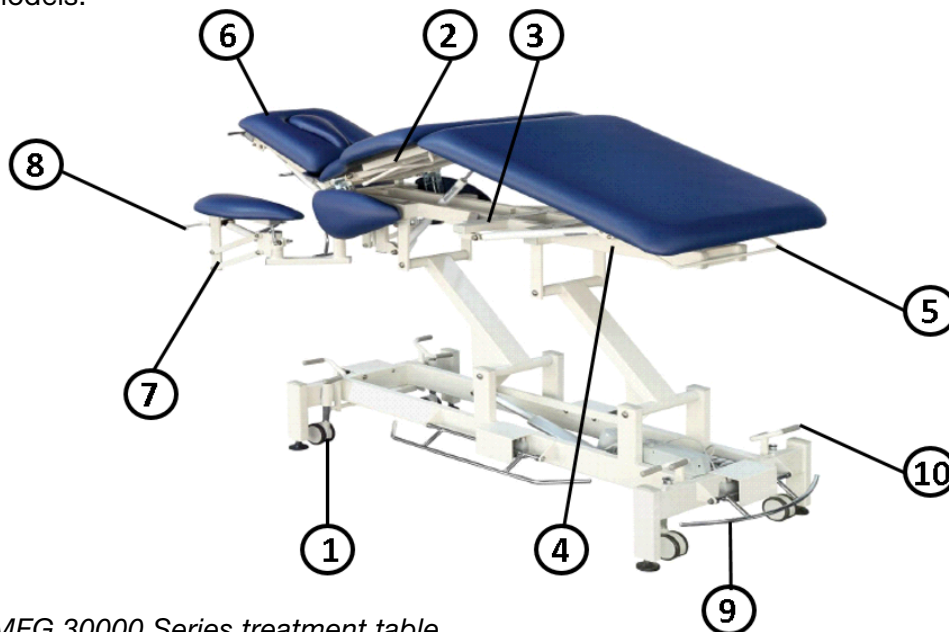


Figure 1. The MFG 30000 Series treatment table

- 1 - Castors
- 2 - Mid section
- 3 - Mid section adjustment lever
- 4 - Leg section
- 5 - Leg section adjustment lever
- 6 - Head section & adjustment lever
- 7 - Arm rest
- 8 - Arm rest adjustment lever
- 9 - Free height adjustment bar
- 10 - Central locking pedal

2. Users Guide

2.1 Inspection upon delivery

Before the device is taken into use, check that the packaging is intact and that it has not been damaged during transportation. Please notify the transport company and the supplier of any transit damage within two (2) days of receiving the delivery.

Ensure that the delivery contains all the parts detailed in the delivery note. If there is anything missing from the delivery consignment, please contact the supplier immediately.

IMPORTANT! Device can be stored at a temperature of 10 to 40°C.
The permitted humidity is 10 to 75%

2.2 Users Guide

IMPORTANT! Do not lift the table from the head section or the arm rests.
Do not push the table into a door sill

The device is intended to be used in normal, dry indoor conditions. Ensure that the temperature of the room is between +10...40°C and the humidity is within the range of 10...75%. If there is chance that device has been exposed to temperatures below 0°C, allow it to adjust to the indoor temperature for at least 5 hours before using any of its features.

Familiarize yourself with the instructions and carry out the following before using the table:

- Make sure that all packing materials have been removed.
- Make sure that the device can freely move up and down.
- Place the device in the location where it will be used and lock the castors.
- Connect the power plug to a socket whose supply voltage corresponds to the voltage shown on the device's type plate. Make sure that the cord runs freely from the connection box.

IMPORTANT! Do not bind the power cord to the device as the lifting motion can sever the cord. Ensure that the cord is easily detachable in an emergency situation.

IMPORTANT! Always detach the power cord before moving the device. Make sure that the cord doesn't get stuck between parts of the frame or under the castors.

IMPORTANT! Make sure that the patient doesn't accidentally move the place free height adjustment bar. As a precaution, detach the power cord when treating the patient.

IMPORTANT! Do not place anything under the device (e.g. chair) because device has to be able to move freely. Make sure that the patient's limbs do not get caught in the frame of the device.

IMPORTANT! Do not place the device under any wall structures or too close to the wall.

IMPORTANT! Do not modify the structure of the device or install parts other than those mentioned in this document.



Warning! Children or people with no experience of the device or those with restricted understanding must not use the device. Children must be supervised to ensure that they do not play with the device!

3. Using the treatment table

Note: Do not use the electrical functions of the device non-stop for longer than the permissible two (2) minutes. Longer continuous use may cause the transformer to overheat. If you use electrical functions non-stop for two (2) minutes, keep to the operating time ratio and do not use any electrical functions for 18 minutes.

3.1 Maximum Inclination

The maximum inclination of sections of different treatment tables can be seen below.

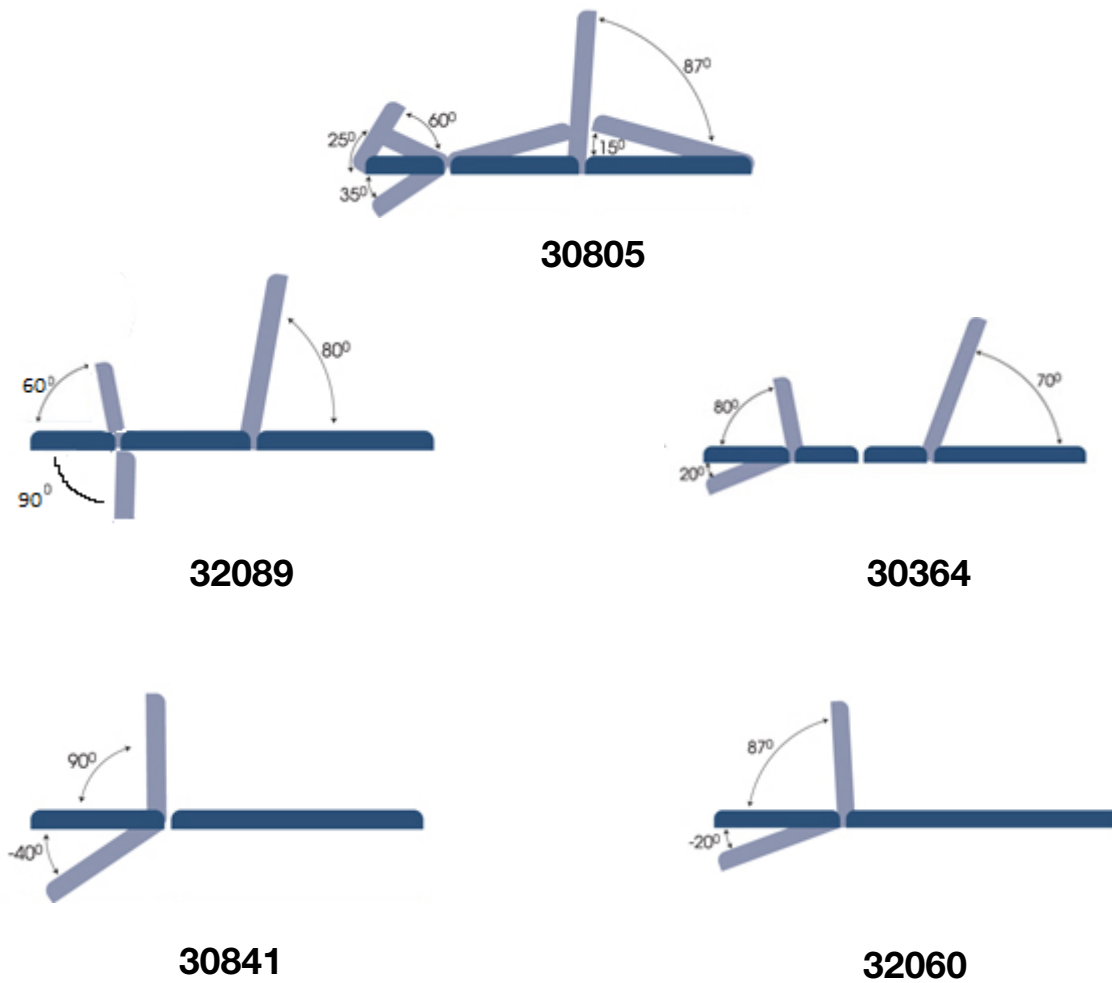


Figure 2. Maximum inclination of the treatment table.

3.2 Locking of the castors

MFG 30000 Series treatment table is fitted with adjustable castors. Lift the table for transportation by pressing the pedal (Figure 3) down.



Figure 3. Locking Panel

IMPORTANT! Always detach the power cord before moving the device. Make sure that the cord doesn't get stuck between parts of the frame or under the castors.

IMPORTANT! Do not use the device for transporting patients.

To lower the table down for treatment, release the pedal.

3.3 Height adjustment

The height of the table is adjusted by moving one of the 4 foot pedals up or down. The height range is approximately 45-91 cm.

Adjust the height by moving anyone of the four foot pedals. To lower the table, lift the foot pedal up. To raise the table, press the pedal down with your foot.

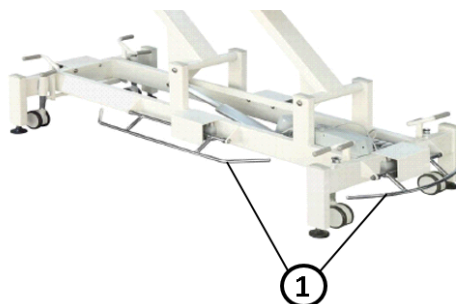



Figure 4. Foot Pedals



Warning! Children or people with no experience of the device or those with restricted understanding must not use the device. As a precaution always detach the power cord when the device is left unsupervised.

3.4 Head Support

MFG 30000 Series tables feature an adjustable head support in most models. To adjust, lift the adjustment lever (figure 6) while lifting/pressing the head support. The adjustment range is shown on page 4 under maximum inclination for various models.

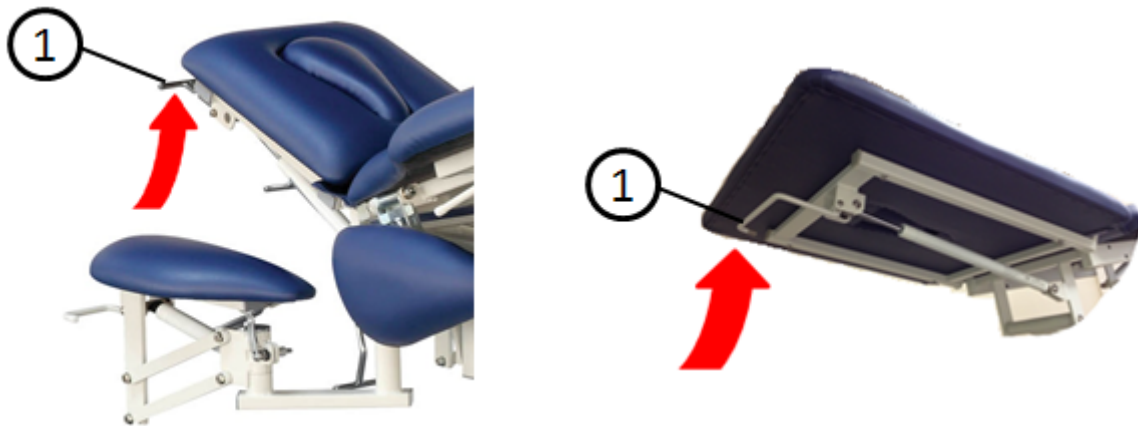

1-Head section adjustment lever

Figure 6. Adjusting the head support

IMPORTANT! Do not lift the device from the head support.

IMPORTANT! Do not sit on the head support. The maximum permissible load on the head support is 10 kg.

3.5 Leg support

The leg support can be raised or lowered by lifting the adjustment lever while lifting/pressing the leg support (figure 7). The adjustment range is shown on page 4 under maximum inclination for various models.



1-Leg support adjustment lever

IMPORTANT! Make sure that nothing gets caught or under the leg support during lifting/lowering

Figure 7. Adjusting the leg support

3.6 Adjusting the mid-section

Lift the table to the drainage position by lifting the adjustment lever (Figure 8). The backrest can be adjusted as shown on page 4 for various models. The angle can be increased by simultaneously lowering the leg support.

Lower from the drainage position by lifting the lever while pressing down on the table top.



1-Mid-section adjustment lever

Figure 8. Adjusting the mid-section

IMPORTANT! Make sure that nothing gets caught or under the middle section during lifting/lowering.

3.7 Arm Rests

Models 30805, 30970 and 32056 have armrests with height and lateral adjustment. To adjust the height, lift the adjustment lever while pressing down on the armrest (Figure 9). When the armrest is at the desired height, release the lever.

Lateral adjustment: Lift the armrest upwards from the point shown by the arrow in Figure 9 and turn the armrest to the side.



1-Armrest adjustment lever


Figure 9. Armrest height and lateral adjustment.

IMPORTANT! Make sure that the armrest is not turned backwards while lowering the device; it may hit the frame and damage the upholstery.

IMPORTANT! Do not sit or lean on the armrest – it cannot take the weight. The maximum permissible load on the armrest is 5 kg.

3.8 Accessories

Accessories for the MFG 30364 Series are shown in Figure 10.

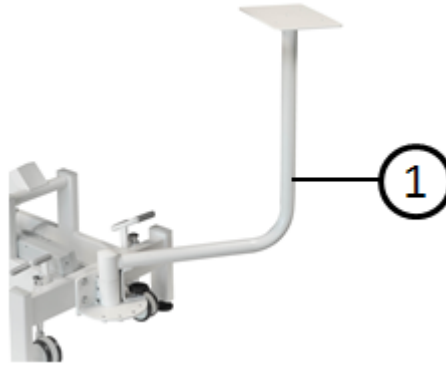

1-Traction unit attachment

Figure 10. Accessories for the MFG 30364